

# Instructions for use

## Empty and disposal of your Pelican Night Drainage:

**Drainable Night Bags:** Simply direct the outlet into the toilet then open the drainage tap (see picture) Always remember to close the tap after emptying and thoroughly rinse between wear. You should change your bag every 5-7 days.


**Non-Drainable Night Bags:** Tear along the strip at the top of the bag, and empty by holding it at the base and pouring into the toilet.

**Disposal:** For disposal please contact your Stoma Care Nurse who will advise on local authority guidelines.


**N.B.** Always wash & dry hands thoroughly prior to emptying or connecting your Night Drainage System.

Product Description	Product Code	Pack Size
Non-Drainable	133100	10
Drainable	133200	10

pelican  
HEALTHCARE

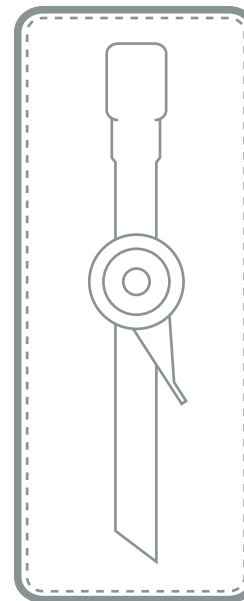

Tap Open

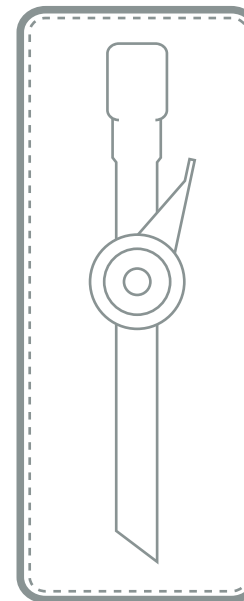

Tap Closed



0800 318 282  
57040 11/16

Pelican Healthcare Ltd,  
Greypoint, Cardiff Business Park, Cardiff. CF14 5WF

pelicanhealthcare.co.uk  
an eakin company