



A guide for Nurses

Pelican
ModaVi

pelican
HEALTHCARE

Pelican ModaVi – bringing you to the forefront

Led by insights, and designed for your patient's optimal wellbeing, **Pelican Healthcare** is rethinking ostomy pouches – for good.

Developed using nurse and patient insights, **Pelican ModaVi** focuses on the features that matter most.

The result?

A new range of ostomy pouches that offer your patients the style, comfort and confidence to be their best selves.

Why Pelican ModaVi?

Practical, comfortable, and stylish, **Pelican ModaVi** ticks all the boxes.

Designed with patients at the forefront, our ostomy pouches offer the confidence of cutting-edge design, the comfort and style of a lifestyle product, and the features that let patients take control of their day-to-day stoma care.

Pelican ModaVi options:

Closed



Pelican ModaVi
Closed Pouch



Flat



Pelican ModaVi
Soft Convex
Closed Pouch



Soft



Pelican ModaVi
Firm Convex
Closed Pouch



Firm



Pelican ModaVi
2PC Closed Pouch

Drainable



Pelican ModaVi
Drainable Pouch



Flat



Pelican ModaVi
Soft Convex
Drainable Pouch



Soft



Pelican ModaVi
Firm Convex
Drainable Pouch



Firm



Pelican ModaVi
2PC Drainable
Pouch

Available in **S** **M** **L**

All pouches available in Neutral or Black
Available in cutting sizes from 12mm up to 80mm

Features & Benefits



- **Shape & Structure:** A new profile shape and weld, ensuring a more natural fit for a more comfortable experience.
- **Colour:** A choice of two stylish colours to offer your patients more discretion.
- **Fabric:** New water resistant fabric, to keep patients feeling fresh for longer.

- **Outlet:** Improved outlet. Wider, simpler and easier to use when emptying the pouch.
- **Filter:** Improved design can help to reduce pancaking and ballooning.
- **Window:** The viewing window enables easy application and monitoring.
- **Fold:** Introducing the fold. For when a more discreet look is needed.



Innovative Fabric

Pelican ModaVi's new softer lightweight fabric has been specially designed to fit into everyday life – so patients can carry on being active, social and confident.

Not only does it provide the look, feel, and comfort of underwear – it also blends the features that matter most for everyday stoma care.

These include:

- Quick-drying
- Watermark resistant
- A lighter material so less noticeable when being worn
- Quiet and discreet
- Softer fabric for maximum comfort
- Stays fresher for longer
- More durability for greater confidence
- Stylish and more attractive
- Enhances everyday wellbeing

'It felt really smooth under my shirt. Normally if you touch a bag through your shirt you get a crinkly sound. Because of this awesome fabric it was made from there was none of that. To be honest most of the time I forgot I was wearing a bag.'

Stylish Colours

Pelican ModaVi is available in two versatile colours that work with patients' lifestyles.



Black – is bold, classic and timeless. It's a stylish pouch that matches many underwear and wardrobe staples – and it's a colour that exudes confidence.



Neutral – a more modern, stylish take on the traditional stoma pouch to gives your patients more discretion so they can feel their best.

'Black just feels less medical and more everyday – like your underwear.'

'Neutral is more discreet, it blends more.'



Iconic Shape

Shape matters – and **Pelican ModaVi** is both iconic and elegant.

Ostomates told us that size and shape were important factors, so we've designed our new pouches with their needs in mind.

Here's how:

- A discreet profile lets patients confidently carry on with everyday activities
- A design that's comfortable whether the pouch is empty or full
- A new shape with discreet notches that quickly drain away water after daily activities such as showering, bathing or swimming

'This feels a bit more elegant, there's shape to it and psychologically it just looks thinner.'

'The shape gives you the illusion that it is smaller than other bags.'



Unique Viewing Window

Pelican ModaVi's larger viewing window gives you more visibility, and gives your patients better control of their stoma care routine.

Discreetly hidden behind an easy-to-open hood, our viewing window is designed to give your patients the confidence to carry on with their daily activities.

Here's why it makes a difference:

- A larger viewing window than other pouches gives better visibility and control
- It's discreetly hidden behind an easy-to-open hood
- Simply 'lift and lock' the hood to see the whole stoma
- This function makes monitoring your patients' stoma and output easier – making it an ideal choice for anyone new to having a stoma
- The larger window helps patients to position their pouch correctly for enhanced comfort and wellbeing
- It lets your patients confidently take control so they can avoid embarrassing leaks and get a good night's sleep

'The hood is good, you've only got to lift it, much better design.'

'The window is so much easier to keep track of things. It's a great concept it means you don't have to keep trying to peel the bag apart and trying to work out what is coming out and what isn't.'

Discretion Fold



Pelican ModaVi gives your patients the flexibility to change the size and shape of their pouch to suit their lifestyle.

Intimate moments, physical activities and certain types of clothing will require more discretion – and this foldability feature lets them adapt their pouch for maximum comfort and confidence.

Here's how:

- It's handy, easy to use and simple to understand
- It avoids unnecessary pouch changes – so reduces wastage
- Its versatility means the size and shape of the pouch can be changed to your patients' lifestyle
- Added discretion boosts comfort and confidence
- It can be easily folded into a pocket or bag
- It lets your patients easily fold their pouch until stoma activity is expected

'I would fold in the summer with a bikini, or when you're being intimate – that would be really good.'

'The way you can secure it completely in half made it feel tiny when I was wearing it, really discreet, you couldn't see it under my clothes. It just made me feel a lot more confident.'


Smart Structure



When it comes to appearance, **Pelican ModaVi's** smart structure gives it the edge.

Here's why:

- Cutting-edge design means the pouch keeps its form, structure and shape – even when filled
- Its smart appearance and iconic shape aim to give patients greater comfort
- A consistent shape gives patients the confidence to carry on with their daily activities

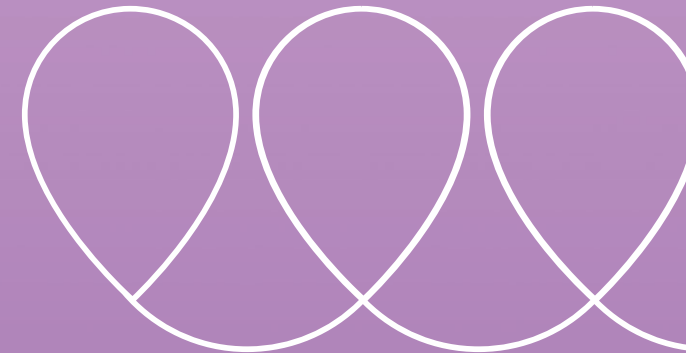

'It really holds its shape well, it's a unique stylish design, it doesn't fill up too wide, it's not hanging down.'

'Bag fills up really evenly.'

Simple Outlet



The **Pelican ModaVi** outlet keeps it simple – so your patients can quickly get back to their daily activities.

Here's how:

- It's easy to open, empty and clean
- A roll-up outlet design features new scalloped edges to aid finger positioning and make opening the outlet even easier
- A notch above the outlet lets any trapped water drain away quickly – making it ideal for daily activities such as showering, bathing, and swimming
- The addition of easy grip 'ears' offer better grip of the Velcro® wings for easier opening and closing – making it an ideal choice for all dexterity levels
- A larger outlet area than other pouches means it's ideal for emptying and cleaning
- Added security gives patients the confidence to take control – with minimum impact on their day-to-day routine

'Outlet is much larger. Tabs are really useful. Very clean don't get any gunk on my hands. Couldn't be happier.'

'Easy cleaning, easy to empty, easy to fold it up again.'

Advanced Filter



Pelican ModaVi's improved filter design means better comfort, greater confidence and less odour.

Here's why:

- Designed with more carbon to help reduce odour – in fact, it has 6 times more carbon than our older pouches
- Potential to reduce pancaking and ballooning
- Designed to aid a better night's sleep
- Designed for superior comfort throughout wear
- It gives patients the confidence to carry on with their day-to-day activities

'The air filter is just amazing. This one works amazingly. I wore it to bed last night and usually every single night without fail I would get up to empty my bag but last night was the first night in years I didn't have to do that. That was amazing, having a good night sleep for once.'

Vitamin E-Infused adhesive baseplate – extra nourishment for better skin health*

6 in 10 patients¹ develop peristomal skin problems in the first 3 months following stoma formation.

Pelican ModaVi pouches* are infused with 1% Vitamin E. The Vitamin E starts to work in 15 minutes of application – and continues to nourish for at least 72 hours.²

Here's how it can make a difference:

- It may help to improve skin condition³
- It helps to moisturise and nourish the delicate skin around the stoma³
- It may help to reduce sore and irritated skin caused by the application and removal of a pouch³
- It may help to improve overall comfort level of the adhesive baseplate³

* Applies to one-piece only

¹ *Salvadarena G (2008) Incidence of complications of the stoma and peristomal skin among individuals with colostomy, ileostomy, and urostomy: a systematic review. J Wound Ostomy Continence Nurs.*

² *In-vitro analysis undertaken using the High Performance Liquid Chromatography (HPLC) technique.*

³ *All statistics, case studies, quotations are based on real world user and/or HCP experiences of using the product as part of an overall stoma care regime. The evaluation did not constitute a randomised or controlled study. All data is held on file by Pelican Healthcare.*



Also available in black

The Pelican ModaVi 2 Piece – a pouch that fits around everyday life

Pelican ModaVi is also available in a 2 Piece which offers all the signature elements of the ModaVi 1 Piece* with added easy-to-use features.

These include:

- A pop-up purple guide ring
- 6 fluted, longer-wear time adhesive baseplate
- Enhanced flexibility and comfort
- The ability to tailor the pouch to suit your patients' lifestyle
- Optimum security for added peace of mind

Pelican ModaVi pouches work seamlessly with our Pelican Platinum adhesive baseplates.





* Excludes Vitamin E

Also available in neutral

S Small **M** Medium **L** Large



Pelican ModaVi 2PC Closed Pouch

Pack size: 30 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|-------------------------------------|----------|-----------|
| P2CS6N | P2CS6B | Pelican ModaVi 2 Piece Closed Pouch | S | 60mm Ring |
| P2CM6N | P2CM6B | Pelican ModaVi 2 Piece Closed Pouch | M | 60mm Ring |
| P2CL6N | P2CL6B | Pelican ModaVi 2 Piece Closed Pouch | L | 60mm Ring |
| P2CM8N | P2CM8B | Pelican ModaVi 2 Piece Closed Pouch | M | 80mm Ring |
| P2CL8N | P2CL8B | Pelican ModaVi 2 Piece Closed Pouch | L | 80mm Ring |

Pelican ModaVi 2PC Drainable Pouch

Pack size: 30 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|--|----------|-----------|
| P2DS6N | P2DS6B | Pelican ModaVi 2 Piece Drainable Pouch | S | 60mm Ring |
| P2DM6N | P2DM6B | Pelican ModaVi 2 Piece Drainable Pouch | M | 60mm Ring |
| P2DL6N | P2DL6B | Pelican ModaVi 2 Piece Drainable Pouch | L | 60mm Ring |
| P2DM8N | P2DM8B | Pelican ModaVi 2 Piece Drainable Pouch | M | 80mm Ring |
| P2DL8N | P2DL8B | Pelican ModaVi 2 Piece Drainable Pouch | L | 80mm Ring |

Platinum 2PC - 60mm Ring Size

Pack size: 5 baseplates

| Product Code | Product Description | Size | Type |
|--------------|-------------------------|----------------|-----------|
| 600015 | Platinum Flat Baseplate | CTF 15-37.5mm | 60mm Ring |
| 600020 | Platinum Flat Baseplate | Pre Cut 20mm | 60mm Ring |
| 600022 | Platinum Flat Baseplate | Pre Cut 22.5mm | 60mm Ring |
| 600025 | Platinum Flat Baseplate | Pre Cut 25mm | 60mm Ring |
| 600027 | Platinum Flat Baseplate | Pre Cut 27.5mm | 60mm Ring |
| 600030 | Platinum Flat Baseplate | Pre Cut 30mm | 60mm Ring |
| 600032 | Platinum Flat Baseplate | Pre Cut 32.5mm | 60mm Ring |
| 600035 | Platinum Flat Baseplate | Pre Cut 35mm | 60mm Ring |
| 600037 | Platinum Flat Baseplate | Pre Cut 37.5mm | 60mm Ring |

Platinum 2PC - 80mm Ring Size

Pack size: 5 baseplates

| Product Code | Product Description | Size | Type |
|--------------|-------------------------|--------------|-----------|
| 800015 | Platinum Flat Baseplate | CTF 15-50mm | 80mm Ring |
| 800040 | Platinum Flat Baseplate | Pre Cut 40mm | 80mm Ring |
| 800045 | Platinum Flat Baseplate | Pre Cut 45mm | 80mm Ring |
| 800050 | Platinum Flat Baseplate | Pre Cut 50mm | 80mm Ring |

Platinum 2PC - Convex 60mm Ring Size

Pack size: 5 baseplates

| Product Code | Product Description | Size | Type |
|--------------|--------------------------------|--------------|-----------|
| 600015C | Platinum Soft Convex Baseplate | CTF 15-40mm | 60mm Ring |
| 600020C | Platinum Soft Convex Baseplate | Pre Cut 20mm | 60mm Ring |
| 600025C | Platinum Soft Convex Baseplate | Pre Cut 25mm | 60mm Ring |
| 600030C | Platinum Soft Convex Baseplate | Pre Cut 30mm | 60mm Ring |
| 600035C | Platinum Soft Convex Baseplate | Pre Cut 35mm | 60mm Ring |
| 600040C | Platinum Soft Convex Baseplate | Pre Cut 40mm | 60mm Ring |

Platinum 2PC - Convex 80mm Ring Size

Pack size: 5 baseplates



| Product Code | Product Description | Size | Type |
|--------------|--------------------------------|-------------|-----------|
| 800015C | Platinum Soft Convex Baseplate | CTF 15-60mm | 80mm Ring |



Style
meets
comfort



Pelican ModaVi Closed Pouch

Pack size: 30 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|-----------------------------|----------|--------------|
| PCS65N | PCS65B | Pelican ModaVi Closed Pouch | S | CTF 12-65mm |
| PCM65N | PCM65B | Pelican ModaVi Closed Pouch | M | CTF 12-65mm |
| PCL65N | PCL65B | Pelican ModaVi Closed Pouch | L | CTF 12-65mm |
| PCL80N | PCL80B | Pelican ModaVi Closed Pouch | L | CTF 15-80mm |
| PCM25PN | PCM25PB | Pelican ModaVi Closed Pouch | M | Pre Cut 25mm |
| PCM30PN | PCM30PB | Pelican ModaVi Closed Pouch | M | Pre Cut 30mm |
| PCM35PN | PCM35PB | Pelican ModaVi Closed Pouch | M | Pre Cut 35mm |
| PCM40PN | PCM40PB | Pelican ModaVi Closed Pouch | M | Pre Cut 40mm |



Pelican ModaVi Soft Convex Closed Pouch


Pack size: 10 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|---|----------|--------------|
| PCSS40N | PCSS40B | Pelican ModaVi Soft Convex Closed Pouch | S | CTF 12-40mm |
| PCSM40N | PCSM40B | Pelican ModaVi Soft Convex Closed Pouch | M | CTF 12-40mm |
| PCSL40N | PCSL40B | Pelican ModaVi Soft Convex Closed Pouch | L | CTF 12-40mm |
| PCSL60N | PCSL60B | Pelican ModaVi Soft Convex Closed Pouch | L | CTF 15-60mm |
| PCSM25PN | PCSM25PB | Pelican ModaVi Soft Convex Closed Pouch | M | Pre Cut 25mm |
| PCSM30PN | PCSM30PB | Pelican ModaVi Soft Convex Closed Pouch | M | Pre Cut 30mm |
| PCSM35PN | PCSM35PB | Pelican ModaVi Soft Convex Closed Pouch | M | Pre Cut 35mm |
| PCSM40PN | PCSM40PB | Pelican ModaVi Soft Convex Closed Pouch | M | Pre Cut 40mm |

Pelican ModaVi Firm Convex Closed Pouch



Pack size: 10 pouches
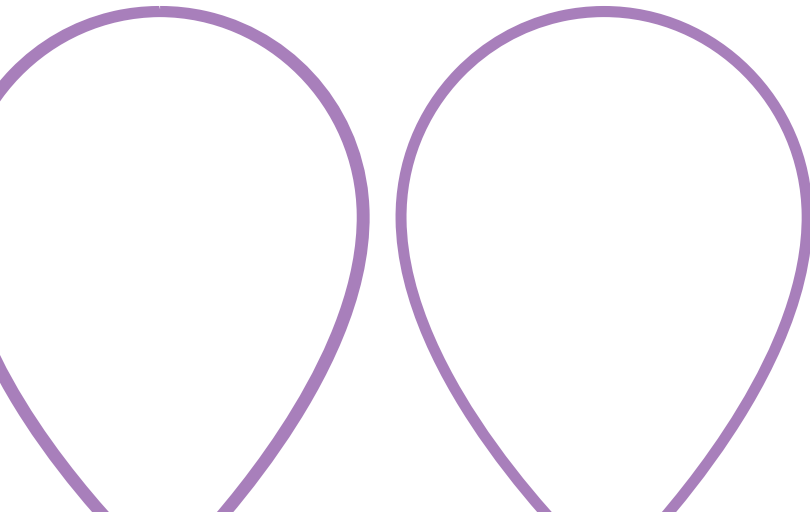
| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|---|----------|-------------|
| PCFS40N | PCFS40B | Pelican ModaVi Firm Convex Closed Pouch | S | CTF 12-40mm |
| PCFM40N | PCFM40B | Pelican ModaVi Firm Convex Closed Pouch | M | CTF 12-40mm |
| PCFL40N | PCFL40B | Pelican ModaVi Firm Convex Closed Pouch | L | CTF 12-40mm |
| PCFL60N | PCFL60B | Pelican ModaVi Firm Convex Closed Pouch | L | CTF 15-60mm |





Pelican ModaVi Drainable Pouch

Pack size: 30 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|--------------------------------|----------|--------------|
| PDS65N | PDS65B | Pelican ModaVi Drainable Pouch | S | CTF 12-65mm |
| PDM65N | PDM65B | Pelican ModaVi Drainable Pouch | M | CTF 12-65mm |
| PDL65N | PDL65B | Pelican ModaVi Drainable Pouch | L | CTF 12-65mm |
| PDL80N | PDL80B | Pelican ModaVi Drainable Pouch | L | CTF 15-80mm |
| PDM25PN | PDM25PB | Pelican ModaVi Drainable Pouch | M | Pre Cut 25mm |
| PDM30PN | PDM30PB | Pelican ModaVi Drainable Pouch | M | Pre Cut 30mm |
| PDM35PN | PDM35PB | Pelican ModaVi Drainable Pouch | M | Pre Cut 35mm |
| PDM40PN | PDM40PB | Pelican ModaVi Drainable Pouch | M | Pre Cut 40mm |


Pelican ModaVi Soft Convex Drainable Pouch


Pack size: 10 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|--|----------|--------------|
| PDSS40N | PDSS40B | Pelican ModaVi Soft Convex Drainable Pouch | S | CTF 12-40mm |
| PDSM40N | PDSM40B | Pelican ModaVi Soft Convex Drainable Pouch | M | CTF 12-40mm |
| PDSL40N | PDSL40B | Pelican ModaVi Soft Convex Drainable Pouch | L | CTF 12-40mm |
| PDSL60N | PDSL60B | Pelican ModaVi Soft Convex Drainable Pouch | L | CTF 15-60mm |
| PDSM25PN | PDSM25PB | Pelican ModaVi Soft Convex Drainable Pouch | M | Pre Cut 25mm |
| PDSM30PN | PDSM30PB | Pelican ModaVi Soft Convex Drainable Pouch | M | Pre Cut 30mm |
| PDSM35PN | PDSM35PB | Pelican ModaVi Soft Convex Drainable Pouch | M | Pre Cut 35mm |
| PDSM40PN | PDSM40PB | Pelican ModaVi Soft Convex Drainable Pouch | M | Pre Cut 40mm |

Pelican ModaVi Firm Convex Drainable Pouch

Pack size: 10 pouches

| Neutral Fabric Product Code  | Black Fabric Product Code  | Product Description | Size | Type |
|---|---|--|----------|-------------|
| PDFS40N | PDFS40B | Pelican ModaVi Firm Convex Drainable Pouch | S | CTF 12-40mm |
| PDFM40N | PDFM40B | Pelican ModaVi Firm Convex Drainable Pouch | M | CTF 12-40mm |
| PDFL40N | PDFL40B | Pelican ModaVi Firm Convex Drainable Pouch | L | CTF 12-40mm |
| PDFL60N | PDFL60B | Pelican ModaVi Firm Convex Drainable Pouch | L | CTF 15-60mm |

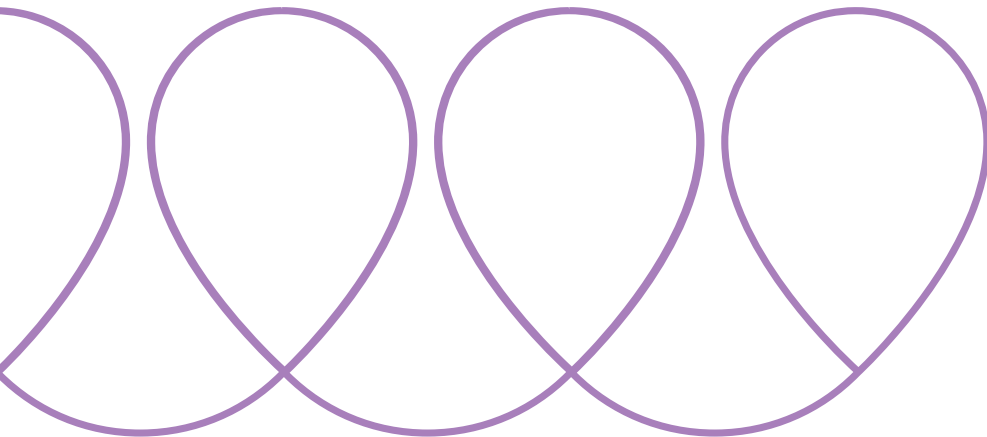

For any additional information:

Call **0800 318 282**

Visit **pelicanhealthcare.co.uk**

Email **contactus@pelicanhealthcare.co.uk**

 PelicanHealthcare

 PelicanHealthcareLtd

 PelicanHealth

 Pelican Healthcare Ltd

PMVBOOKN/1



pelican
HEALTHCARE