


# Instructions for use

High Output

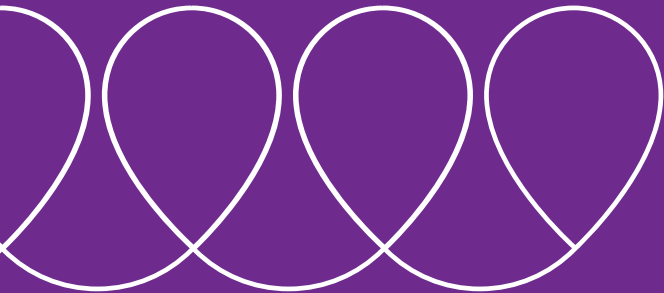

pelican  
HEALTHCARE


# About this product

The enclosed Pelican 1-piece High Output pouches are used to collect output from a stoma. The baseplate attached to the pouch is intended to adhere to intact peristomal skin. Individual wear times may vary.

A convex option may be suitable for retracted, flush or other challenging stomas only after assessment from a stoma care professional.


Our Pelican High Output pouches are compatible with Faecal Collector pouches.

# How to apply your pouch

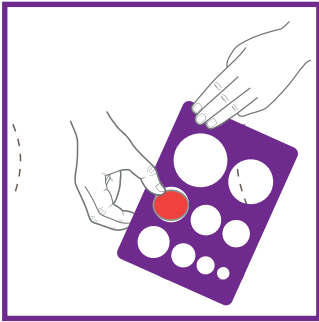

## 1. Choosing the correct size pouch


If your stoma is round then your Pelican pouch may be available in pre-cut sizes which will give you a snug fit. Use the enclosed measuring guide and choose a hole size which is the same, or slightly larger than your stoma.

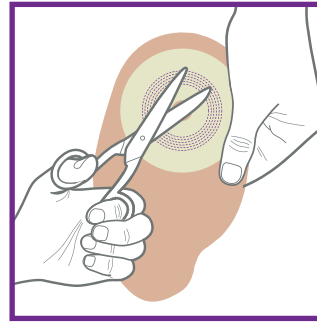

## 2. Cutting the baseplate (on cut-to-fit pouches only)


If your stoma is not round or you wish to cut a particular circle, trace the shape onto the release film and cut with a curved scissors, taking care not to cut through the pouch film on the other side. An accurate fit reduces the risk of leaks and skin problems.

# How to remove your pouch

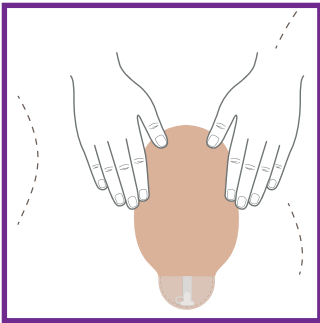


### 3. Applying the adhesive baseplate

First prepare skin by ensuring the area around your stoma is clean and dry. Remove the release film. Fold back the top of the pouch and locate by placing the lower part of the baseplate below the stoma. Now press the baseplate against your skin and smooth it with your fingers.

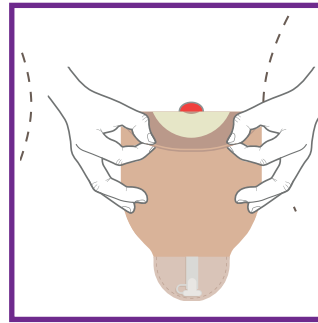


Gently remove the pouch by peeling the baseplate downwards and away from the skin. Apply light pressure to your skin with your free hand to aid removal.


# How to drain your pouch

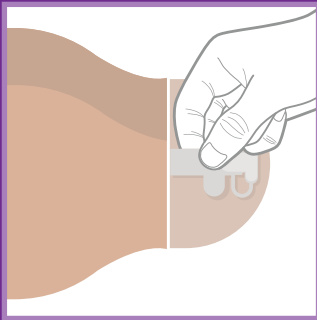


**1.** Gently squeeze the outlet to stem the flow of output and then release the bung stopper.

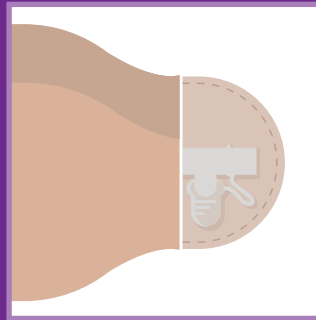

**2.** Fold back the stopper and connect to the bung rest. The bung rest is designed to hold the stopper clear of the outlet to aid pouch draining.

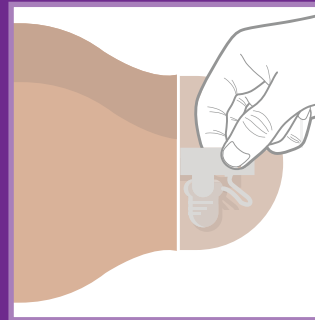


**3.** Release the outlet to allow the output to pass from the pouch. Once empty, wipe the outlet clean and replace the bung stopper until secure.

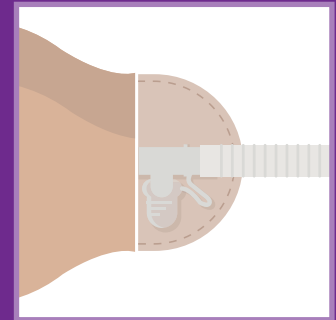

**4. Optional**  
To connect a faecal collection bag, squeeze the outlet again and simply connect the tube directly over the bung. Do not replace the bung stopper.

# How to use the filter covers

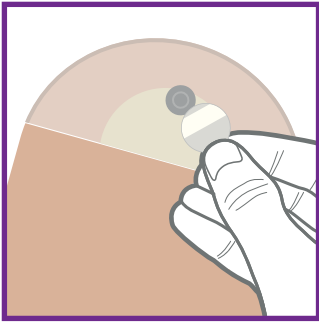


These can be used on our high output pouches and are normally used when bathing or to prevent 'pancaking' (when the firmer stool stays around the stoma and does not drop into the pouch). A filter cover can be applied by lifting up the split fabric and placing the cover over the filter which can be seen as a dark circle. Remember to remove filter cover once air level has balanced or once finished bathing/showering.

# Disposal



Place used products into waste bag and waste bin.

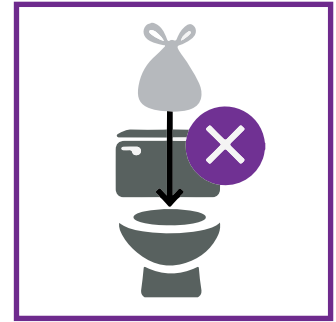

Do not flush product down the toilet. Dispose of waste products in accordance with local guidelines.

# Warnings and cautions

Consult a healthcare professional if any skin irritation, redness or discomfort around the stoma is experienced. Re-use of a single use product may have potential to harm the user due to the risk of contamination and impairment in product function.

Skin adhesion and wear time may be reduced in conditions of elevated temperature or humidity, or if the baseplate is re-positioned after first application.

In order to achieve correct fitting ensure the stoma template is large enough and the bag is applied as instructed. Incorrect fitting may lead to skin damage e.g. broken skin, rash, irritation and potential infection.

## Other information

Store flat in the carton between 10° and 30° Celsius. Do not refrigerate. Keep dry. Do not use if packaging damaged.

Pelican Healthcare Ltd accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to Pelican Healthcare Ltd current recommendations. Any serious incident encountered, in relation to the use of this device, should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.



Temperature limit



Keep dry



Do not reuse



Consult instructions for use



Medical Device



Do not use if package is damaged



IFU paper recyclable



Manufacturer



Authorized representative in the European Community/ European Union

**pelican**  
HEALTHCARE



## Pelican Healthcare Ltd

Greypoint, Cardiff Business Park,  
Cardiff, CF14 5WF



## Eakin B.V.

Medelse Poort, Biezenwei 23,  
4004 MB Tiel, Netherlands

For any additional information:

Call **0800 318 282**

Visit **[pelicanhealthcare.co.uk](https://pelicanhealthcare.co.uk)**

Email **[contactus@pelicanhealthcare.co.uk](mailto:contactus@pelicanhealthcare.co.uk)**

 PelicanHealthcare

 PelicanHealthcareLtd

 PelicanHealth



© 2022 Pelican Healthcare Ltd



S7029 Rev 3  
Revision Date: 05/2022

